


PenGuide Assistant for Convenience and Safety

Tailor made to fit your pen system

Device - Innovation


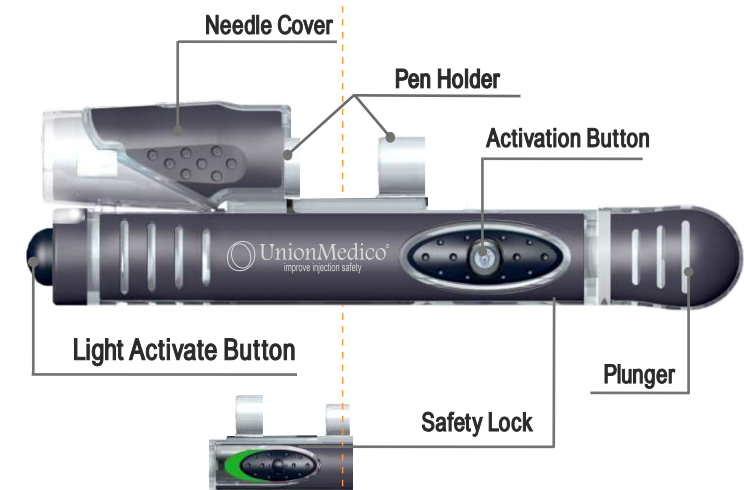


Patent, Trademark and Design Protected Worldwide

Giving You Extraordinary Injection Freedom !

- Remove the needle Phobia/ Anxienty.
- Ensure stability at needle penetration / Super Grip
- Less injection pain.
- Night vision
- Easy and smooth injection
- Smart grip - super grip
- Stabilize injections
- Non-slip rubber grip ergonomic designed handle Grip

Kills Phobia /Anxienty

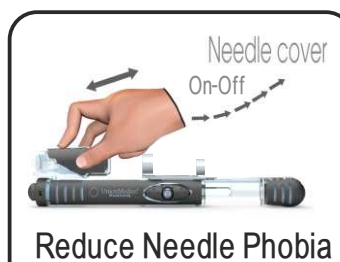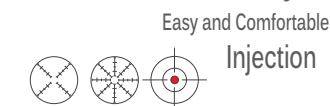

Before use

- ▶ Prepare your Pen according to the instruction for use.




Obs !!!! Keep the inner needle cap on!

- ▶ Carry out ! some tests before injecting. Load / Unlock / Release



- ▶ Slide the plunger backwards until you hear a click.

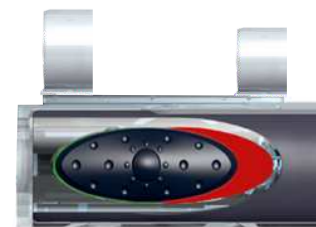

- ▶ The safety lock now shows a red field, and the injector is secured.



- ▶ To unlock the injector Slide the safety lock forward to green exposure.
- ▶ Press activation button to release the plunger.

1 Loading

- ▶ Slide the pen into the injector's holder.



- ▶ 1-When you hear click, the pen is secure in the holder.



2

- ▶ 2- Remove the inner needle cap.

- ▶ 3- Slide the plunger backwards until you hear a click.



- ▶ The Assistant is now ready for injection.

Do not touch the Activation Bottom when you have unlocked the safety slider.


Click on!



2 Prepare Injection


- ▶ Prepare the injection site and use the injection technique recommended by your healthcare professional.

Hold PenGuide against your skin with led light ON!

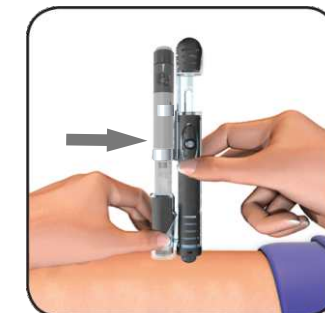

- ▶ Place the PenGuide Assistant at the chosen Injection site.


Do not touch the Activation Bottom when you have unlocked the safety slider.

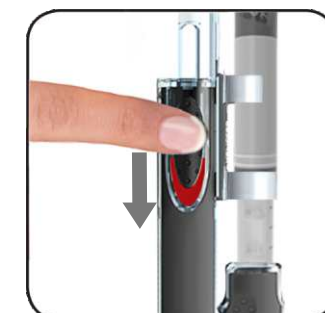

3 Preparing your injection


- ▶ Injection Zone : Abdomen or front of thigh? Pinch or not pinch. According to your [Healthcare Provider!](#)
- ▶ Clean your skin with an alcohol swab where you intent to make the injection.

4 Placing the injector on the skin

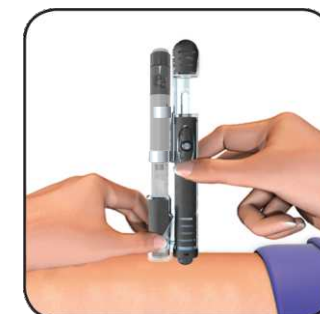

- ▶ Rest the injector gently at the injection site at an 90° angle. The LED light will turn on. **Do not inject yet.**
- ▶ Hold the base of the injector with your non-injection hand to make it steady. The pen should be on the opposite side of your hand.

5 Activating

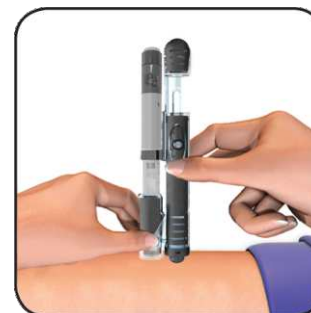

- ▶ Use your index finger and thumb of your injection hand. Push down the safety lock until it stops. The red indication will change to green. Hold the lock in this position with agentle pressure.
- ▶ You have now activated the injector.


Turn Over ▶




- ▶ You have now activated the injector.
- ▶ Place your **thumb** and index finger on the activation **button**.



- ▶ When you are ready, press the button to release the pen and let it slide toward your skin.
- ▶ **Do not hold the pen holder. Your hands must be placed as shown in the picture to make sure that the PenGuide works properly. Do not place your hand, so it can obstruct the motion of the pen holder.**

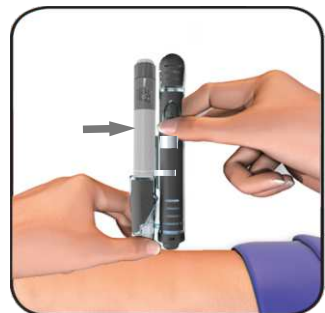


6 Giving yourself the dose

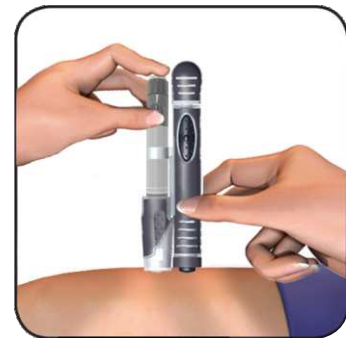

- ▶ How to deliver the dose, see your pen instruction for use.

7 Removing Pen

- ▶ Still holding the base of the PenGuide, slide the pen upwards until you hear a click.

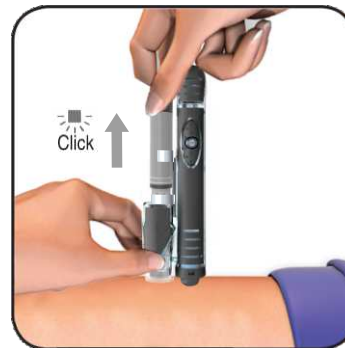

- ▶ Then disconnect pen from PenGuide Assistant.



8 Storing and cleaning



- ▶ To clean the injector, wipe it with a soft cloth moistened in a mild soap solution.
- ▶ Replace the injector in its case until your next injection.

Specifications

Subcutaneous

PenGuide Reusable Device

Dimensions 145 mm x 38 mm

Needle range 4 – 16mm

Lithium Cell 3V



Quality and Compliance

- ▶ Tested and verified as a High functional Injection system ISO 9999 classification 04.19.12
- ▶ Tested according relevant parts of EN 12182
- ▶ Tested at an accredited test institute. A constant impact velocity and a life time of 2 years were documented.
- ▶ CE-marked.
- ▶ ISO 13485 certified and audited according to GMP.

Quality of care for Quality of life

261792A05

2019/03PI120001

Union Medico
Cobis Copenhagen Bio Science Park
Ole Maaløes Vej 3
DK-2200 Copenhagen
DENMARK
www.unionmedico.com

Product Info:

www.penguide-assistant.com

NOTES:

Device
That
Change
Your
Everyday
Life

Your
PenGuide[®] Assistant